



## FlexSupport Manual



### Conformity

This device complies with the requirements of Regulation (EU) 2017/745 Medical Devices (MDR), Class I.



## Introduction

Thank you for choosing EVIBO's FlexSupport.

This pillow is designed to provide lateral support and comfort. It is manufactured with a PU cover, a plastic back structure, and PU foam for durability and comfort. The cushion is suitable for wheelchairs, relax chairs, home armchairs, chairs and other seating facilities.

## Intended use

The FlexSupport is intended for use as a lateral support cushion for individuals with limited trunk balance. It supports the sides of the torso when sitting on a seating facility.

## Do not use this product with:

- Active pressure ulcers or severe skin irritations on the back or flanks
- Severe spine disorders without consulting a practitioner
- Known allergy to polyurethane (PU) or leatherette
- Persons over 120 kg body weight
- Situations in which the cushion may shift due to uncontrolled movements

## Productcomponenten

- Back part with 2 adjustable click buckles (back anti-slip and front Velcro).
- Pelot right (bottom Velcro).
- Pelot left (bottom Velcro).

## Manual

1. Attach the back part to the desired backrest by means of the click buckle and tighten the straps. This can be done vertically or diagonally. See what best fits the back of your chair.



2. Position the pads with the Velcro side on the backrest between the hips and armpits.

**NOTE: The gradually sloping side of the pads should point towards the armpit.**



3. Adjust the cushion so that it fits snugly on both sides of your torso. Make sure the wings embrace your sides for extra support.



## Cleaning

- Remove the PU cover by opening the zipper.
- Wash the cover in the washing machine with a mild detergent at a maximum of 60°C.
- Clean the plastic backing structure and PU foam with a damp cloth.

## Safety

### ⚠ Important:

- Do not use the pillow as a toy.
- Incorrect placement can lead to insufficient or incorrect support.
- A loose-fitting cushion can shift and cause instability.
- Prolonged use without postural adjustment may cause pressure points.
- Poor cleaning can lead to the accumulation of bacteria or allergens.
- Avoid exposure to extreme temperatures.
- Keep the product away from sharp objects.

## Possible side effects

- Pressure points during prolonged use without a change of posture.
- Skin irritation due to sensitivity to materials used.
- Reduced stability when attached incorrectly.

## Maintenance

- Check regularly for signs of wear or damage.
- Replace the cushion in case of loss of shape or damaged cover.
- The expected service life of the product is 2 years with normal use.

## Storage and conditions of use

- Store at room temperature between 5°C and 40°C.
- Protect from moisture and direct sunlight.

## Manufacturer and contact information

### Trademark owner and responsible for marketing in the EU:

Address: Evibo Medical  
Rijksweg 105953 AE Reuver, The Netherlands

Website: [www.evibo.nl](http://www.evibo.nl)

Email: [info@evibo.nl](mailto:info@evibo.nl)

## Product details:



FlexSupport



MODEL: PU491840 Butterfly Embracing Wheelchair Lumbar Cushion



2025-06-07



20250607



120kgs



Company: Dongguan China Effort Healthcare Products Co., Ltd.  
Address: 39th Jinyuan Road, Liaobu Town, Dongguan City Guangdong  
Province, China



SUNGO Europe B.V.  
Fascinatio Boulevard 522, Unit 1.7 2909 VA Capelle aan den IJssel  
Nederland



(01)8720892657404(11)250607(10)20250607